



CARE PUMP

# MOVE 8<sup>PRO</sup>

Advanced line


<b>Number of chambers</b>	8
<b>Max. pressure [mmHg]</b>	220
<b>Number of proprietary therapeutic pre-programs</b>	15
<b>Smart gradient' function</b>	(mmHg, %, final pressure)
<b>Hold function [s]</b>	1-50
<b>Interval function [s]</b>	1-50
<b>Precise regulation of pressure and time</b>	in steps of 1, 5 or 10
<b>Setting of individual pressure in each chamber</b>	included
<b>Number of independent channels</b>	1
<b>Adjustment of the speed of the treatment</b>	1-5
<b>Cuffs</b>	overlapping system
<b>User's own programs</b>	included
<b>List of "Favourite" programs</b>	included
<b>Display</b>	7" TN lcd color touch screen
<b>Weight [kg]</b>	2,9
<b>Dimensions [mm]</b>	290x130x130
<b>Power supply</b>	battery 21VDC/1A
<b>Estimated battery life from full charge</b>	4 h*
<b>Warranty</b>	1 year

\*test carried out using the "Prevention, relaxation" program and the pressure of 100 mmHg

# Accessories




## Included:

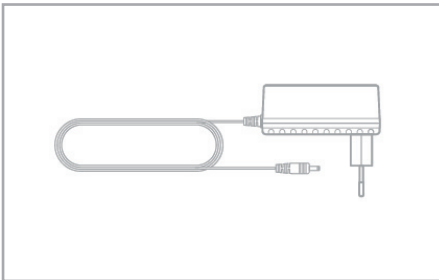


power supply

## Optional:

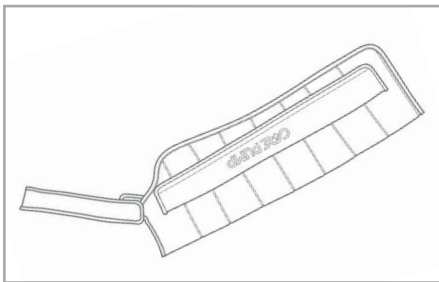


8-chamber arm cuff

CPARM8T

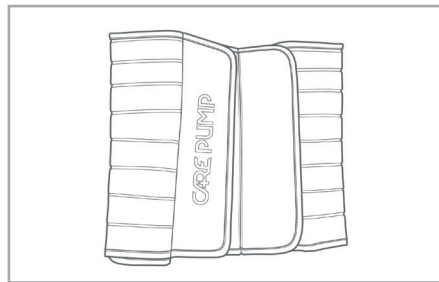


8-chamber waist cuff

CPWAIST8T

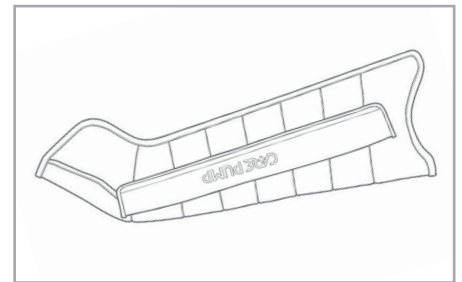


8-chamber left leg cuff

CPLEG8TL

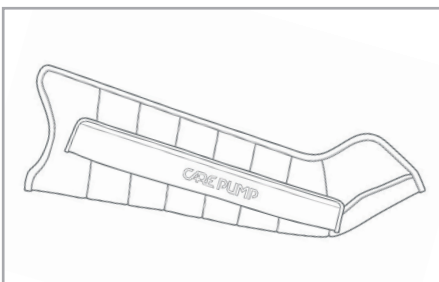


8-chamber right leg cuff

CPLEG8TR

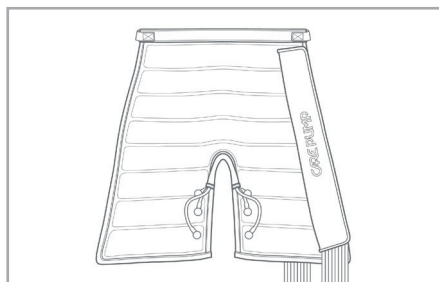


8-chamber short pants

CPSHORTPANTS8T

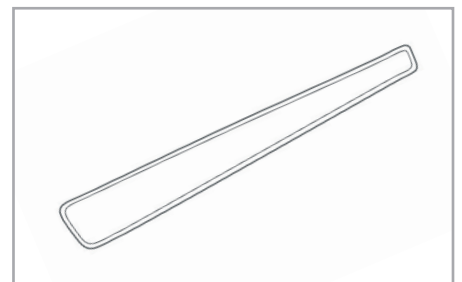


Extension zipper

CPLEGEXT8

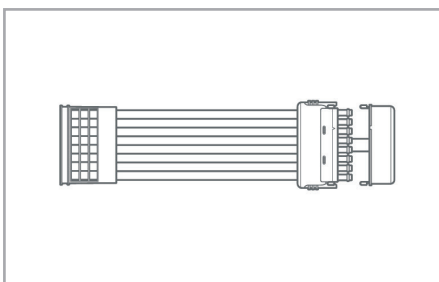


Double splitter

CPDOUBLECONNECTOR8

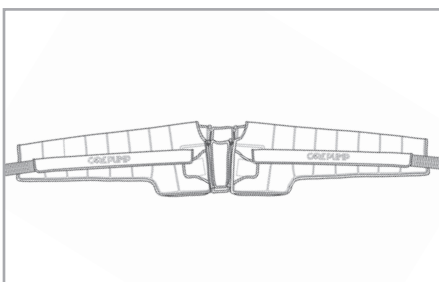


8-chamber double cuff for arms, shoulders and chest

CPDOUBLEARM8T

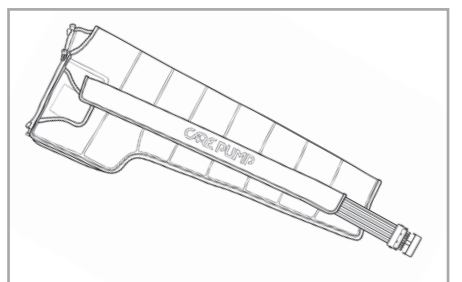


8-chamber left arm, shoulder and chest cuff

CPARMSB8TL

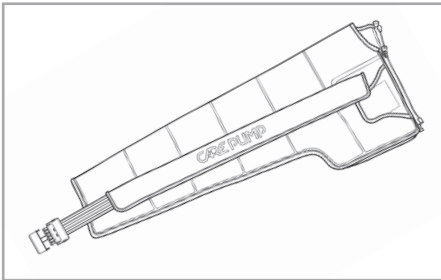


8-chamber right arm, shoulder and chest cuff

CPARMSB8TR

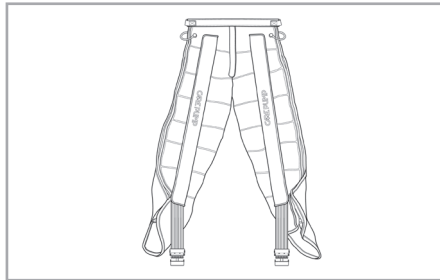


8-chamber full pants

CPLEG8TL

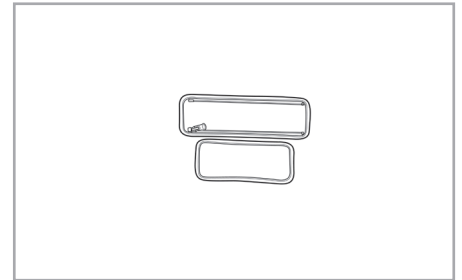


2 x Extension zippers for 8-chamber double cuff for arms, shoulders and chest

CPDOUBLEARMEXT8

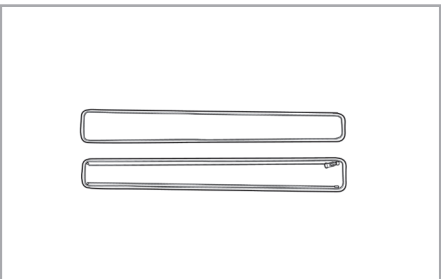


2 x Extension zippers for 8-chamber short pants

CPSHORTPANTSEXT8

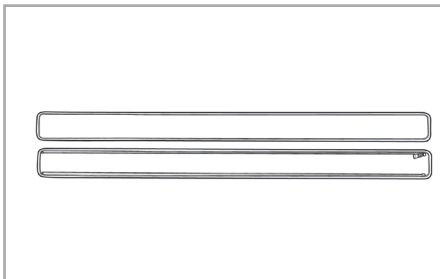

2 x Extension zippers for 8-chamber full pants

CPLONGPANTSEXT8

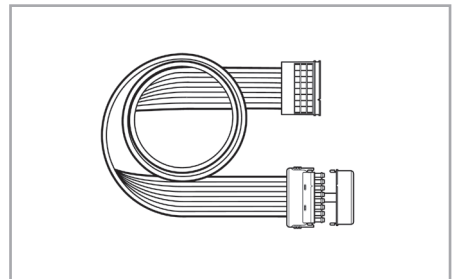

Double splitter (long - 150 cm) for 8-chamber devices

CPDOUBLECONNECTOR8L

# Therapeutic pre-programs



lymphoedema I

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	1 mmHg
<b>Time</b>	50 min.
<b>Hold</b>	3 s
<b>Interval</b>	3 s

lymphoedema II

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	1 mmHg
<b>Time</b>	50 min.
<b>Hold</b>	3 s
<b>Interval</b>	3 s

lymphoedema III

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	1 mmHg
<b>Time</b>	50 min.
<b>Hold</b>	3 s
<b>Interval</b>	0 s

lymphoedema IV

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	1 mmHg
<b>Time</b>	50 min.
<b>Hold</b>	3 s
<b>Interval</b>	0 s

venous oedema I

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	3 mmHg
<b>Time</b>	45 min.
<b>Hold</b>	2 s
<b>Interval</b>	3 s

venous oedema II

- 
- 
- 
- 
- 
- 
- 
- 

<b>Gradient</b>	3 mmHg
<b>Time</b>	45 min.
<b>Hold</b>	2 s
<b>Interval</b>	2 s

lipoedema I

- 
- 
- 
- 
- 
- 
- 
- 


<b>Gradient</b>	2 mmHg
<b>Time</b>	60 min.
<b>Hold</b>	3 s
<b>Interval</b>	3 s

lipoedema II

- 
- 
- 
- 
- 
- 
- 
- 


<b>Gradient</b>	2 mmHg
<b>Time</b>	60 min.
<b>Hold</b>	3 s
<b>Interval</b>	3 s

post-injury therapy

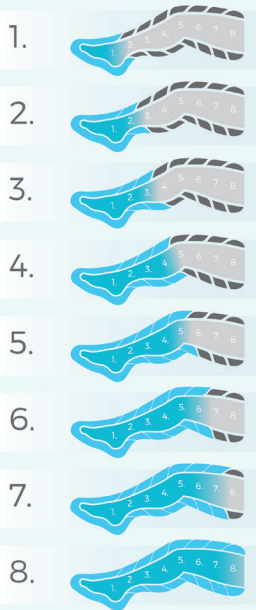


<b>Gradient</b>	4 mmHg
<b>Time</b>	30 min.
<b>Hold</b>	2 s
<b>Interval</b>	2 s

body firming

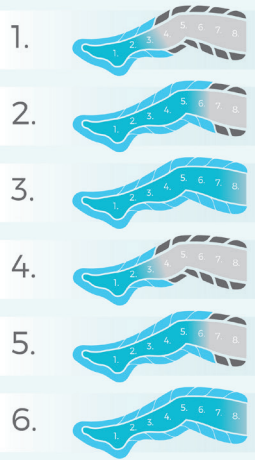


<b>Gradient</b>	1 mmHg
<b>Time</b>	60 min.
<b>Hold</b>	3 s
<b>Interval</b>	3 s

cellulite **advanced**

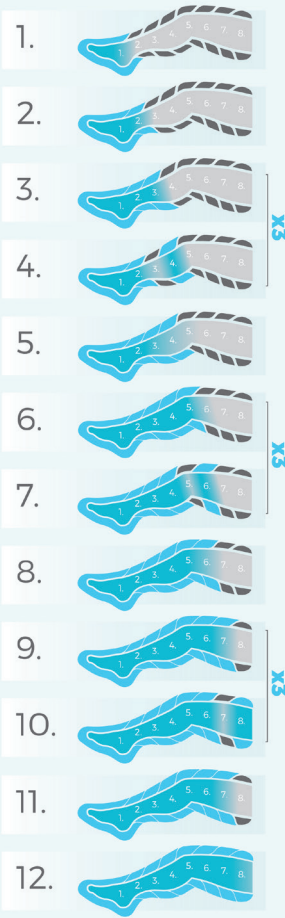


<b>Gradient</b>	1 mmHg
<b>Time</b>	60 min.
<b>Hold</b>	1 s
<b>Interval</b>	2 s

regeneration

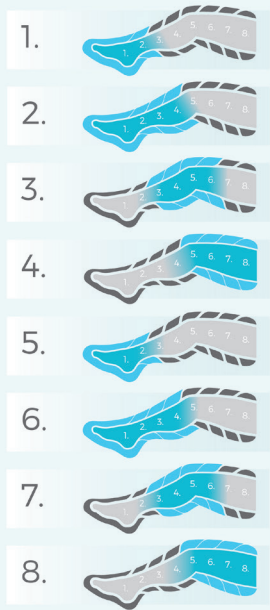


<b>Gradient</b>	4 mmHg
<b>Time</b>	45 min.
<b>Hold</b>	2 s
<b>Interval</b>	2 s

prevention, relaxation

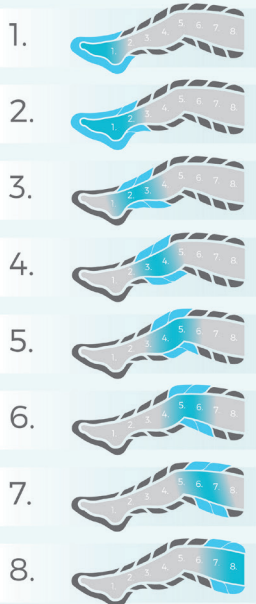

<b>Gradient</b>	0 mmHg
<b>Time</b>	60 min.
<b>Hold</b>	2 s
<b>Interval</b>	2 s

all-chamber massage

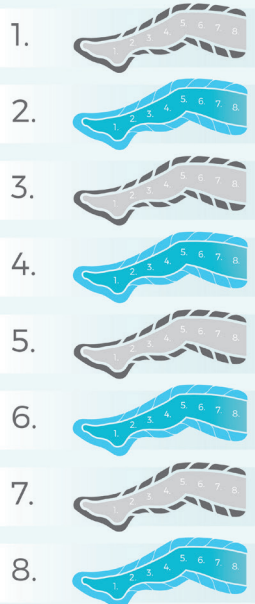

<b>Gradient</b>	0 mmHg
<b>Time</b>	30 min.
<b>Hold</b>	2 s
<b>Interval</b>	3 s

mixed mode

**Possibility of combining any therapeutic pre-programs and selecting the number of repetitions for each of them.**